

INSTRUCTION FOR USE
SECURE T
TOTAL SURFACE DISINFECTANT
BACTERICIDAL- VIRUCIDAL- FUNGICIDAL
READY TO USE

COMPOSITION

Isopropyl Alcohol 25%, Benzethonium Chloride 0.28%, Sterile Distilled Water, Flavours and Stabilizers

EFFECTIVE SURFACE DISINFECTANT AGAINST

Mycobacterium Tuberculosis, Staphylococcus Aureus, Streptococcus Aureus, Hepatitis B virus, Hepatitis C virus, Herpes Simplex type 1 and 2, HIV-1, Covid-19.

PROPERTIES

Benzethonium chloride exhibits a broad spectrum of microbiocidal activity against bacteria, fungi, mold and viruses. This compound is an odorless white solid, soluble in water. It has surfactant, bactericidal, virucidal and fungicidal in nature. Isopropyl Alcohol 25% is powerful total surface disinfectant (Aldehyde and Chlorhexidine Gluconate free, so do not stain on surface).

INDICATIONS

Secure T is a multipurpose disinfectant/decontaminant cleaner for use on Hard, Non-porous, Intimate surface, Hospital furniture, Isolation areas, Instruments (non-corrosive), Surgical operatory, Dental operatory, Incubators, Exterior surface of Dental Curing Light and Ultrasonic Scaler, Intraoral Camera, Exterior Surface of Radiographic Sensor, Glass surfaces, in Ultrasonic Cleaner as Pre Cleaning Procedure.

CONTRAINDICATIONS

Drug idiosyncrasy
Do not misuse

SIDE EFFECTS

Do not introduce directly on human body.

Personal Protection wear appropriate protective gloves, gowns, masks and protective eye goggles. If inserted in eyes, hold eye open and rinse slowly and gently with water for 15-20 minutes.

As far as all terms of storing, transporting and application are observed, there are no side effects

METHODS OF USE

- Surface Disinfectant, Cleaning

Secure T will effectively clean and disinfect, when sprayed directly on surface after inserting trigger sprayer, wipe it after 3 minutes with sterile cloth or tissue paper. Liquid will not evaporate.

- Precleaning Instruments

Secure T will be effective to allow complete submersion, cover and allow to soak for 10 minutes in sterile container, after rinse it properly (Non Corrosive property). Add one ounce of Secure T per liter of water to ultrasonic cleaner, activate solution for 10 minutes, rinse and wash thoroughly under running water and then go with autoclave procedure.

- Wiping Instruments

For Endoscopic instruments, Dental Ultrasonic scaler, Dental Light Cure Unit, Operatory light and other radiographic sensors, spray and leave solution for 3 Minutes and wipe with sterile cloth or tissue paper.

PRESENTATION

Secure T (bottle) 500ml 007-01-500

Trigger sprayer-1

STORAGE

Store at the temperature from 5°C to 25°C. Keep in dry place

Tightly close container immediately after use

Do not use after expiry date

Shelf life – 2 years

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com



info@safeendo.org

SafeEndo
enhance your endo



+91 79-22139977 / 22163335



www.safeendo.org
www.safeendoindia.com



info@safeendo.org