The HANAU™ Spring-Bow
Instruction Manual

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The HANAU™ Spring-Bow

The HANAU™ SPRING-BOW is an Earpiece type, used to capture the record of a patient's maxillary arch and its relation to the external auditory meatus. It is capable of preserving this clinically acceptable relation and to transfer it onto a HANAU™ Articulator.

AUTOCLAVABLE...

SELF-CENTERING...

COMFORTABLE...

RAPID

Uses the ANTERIOR PATIENT REFERENCE of the INFRA-ORBITALE NOTCH and the POSTERIOR PATIENT REFERENCE at the PORION, the superior border of the EXTERNAL AUDITORY MEATUS.

Both references form the anatomical FRANKFORT HORIZONTAL PLANE; the plane upon which this HANAU Spring-Bow is based.

The patient related Spring-Bow suspends DIRECTLY from the Articulator for attaching the maxillary cast or—

the Spring-Bow may remain in the operatory and the maxillary cast attached INDIRECTLY by using an accessory Mounting Platform No. 010356-000.

The Item No. 010328-0 HANAU™ Spring-Bow, Figure 1, includes:

- (1) Bow, Part No. 010355-000
- (1) Transfer Clamp Assembly, Part No. 01356-000
- (1) Anterior Elevator, Part No. 010358-000
- (1) Bitefork, Part No. 005751-000

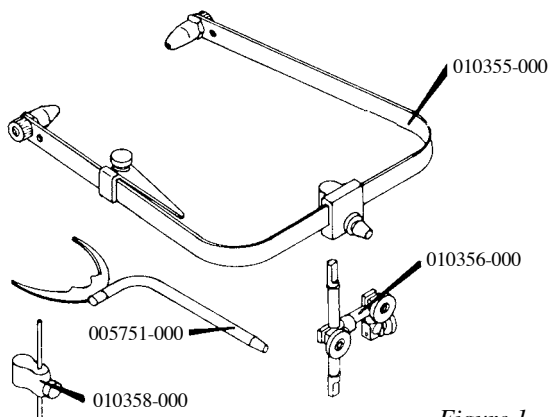
Figure 1

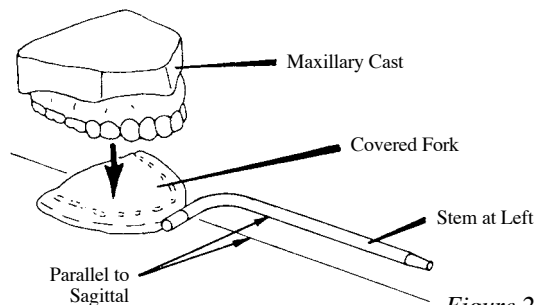
Figure 2

BITEFORK PREPARATION

DENTULOUS - The Bitefork is covered with a triple layer of hard baseplate wax or compound, suitable for obtaining an occlusal imprint of the maxillary cusps, Figure 2.

The softened impression material on the Bitefork is seated against the maxillary occlusal surface to create a distinct cuspal imprint without metal contact. Note that the stem of the Bitefork must extend approximately parallel to the sagittal plane and be at the left side of the patient.

Chill the wax occlusal imprint and check it against the maxillary cast for accurate seating. Relieve any tissue bearing surface which may interfere.

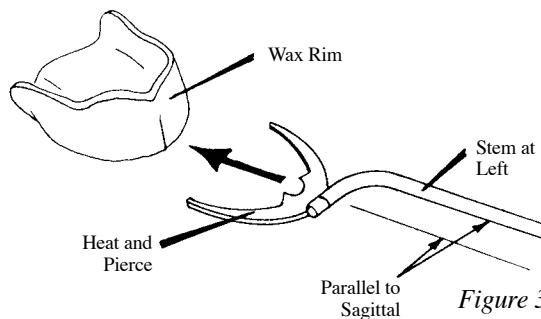
Figure 3

EDENTULOUS - Attach the Bitefork to the maxillary occlusal rim by heating the tines and piercing them fully into the wax rim, Figure 3.

The "tines" shall be parallel to the occlusal plane, perhaps 3 mm from the occlusal surface and shall not distort the occlusal surface. The stem of the Bitefork shall be at the patient's left and be approximately parallel to the sagittal plane, Figure 4. The assembly shall form a rigid occlusal rim/bitefork stem assembly, as though in one piece.

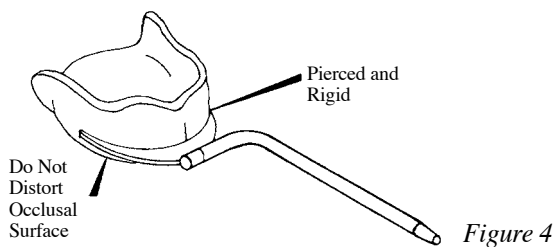
Figure 4

An alternative to piercing the wax rim is to fabricate an occlusal index on the Bitefork.

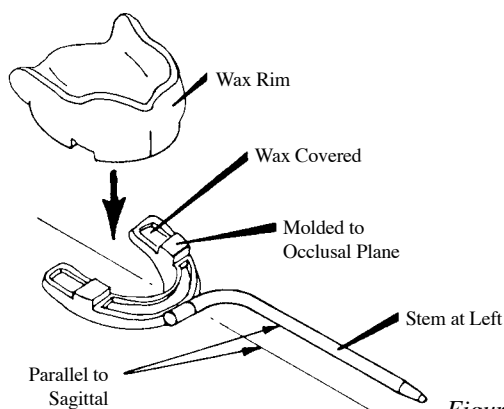
Figure 5

Figure 5 illustrates the Bitefork stem at the patient's left, covered with a triple layer of hard baseplate wax or whatever your selection. Heat seal the periphery and soften throughout in a water bath. Seat and hand mold the wax around the rim and chill to assure removal and accurate replacement of the wax rim in its indexed bite.

BOW PREPARATION

1. Finger attach the Earpieces to Bow using the knurled Thumbnuts, Figure 6. No wrenches are necessary with the HANAU™ Spring-Bow.
2. Note that the Thumbscrew is finger tightened to retain the Anterior Patient Reference in the non-hazardous position illustrated.

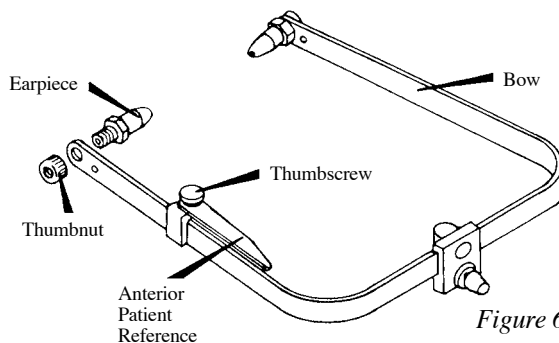
Figure 6

3. Insert the Transfer Rod into the Bow Socket, Figure 7. Align the "flatted" side toward the front and insert to the Rubber Ring "stop." Finger tighten the tapered Thumbscrew against the "flatted" side of the Transfer Rod.

Note the position of Clamps, illustrated.

4. Loosen the Thumbscrews at No. 3 and 2 on the Bitefork Clamp and then Thumbscrew No. 1 on the Transverse Clamp.

A light application of petroleum jelly on the Transfer Rod and Transverse Rod will assist Clamp positioning during patient application.

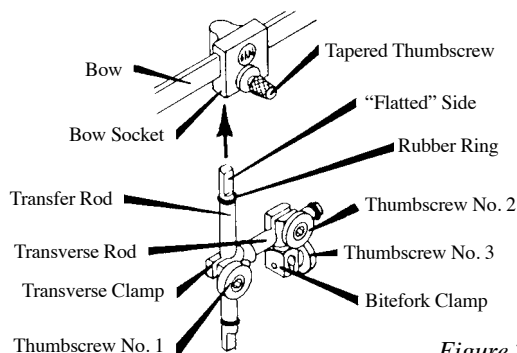
Figure 7

PATIENT APPLICATION

5. Accurately seat the prepared Bitefork to the patient, perhaps placing two cotton rolls between the mandibular occlusal surface and the underside of the Bitefork, Figure 8. Apply a "touch" of petroleum jelly on the stem end of the Bitefork.

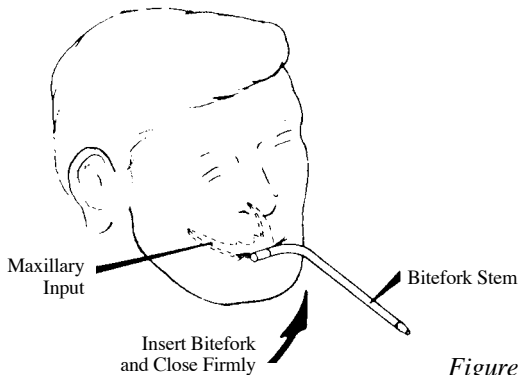
Figure 8

Instruct the patient to gently close and firmly support the Bitefork in this accurate and immobile maxillary occlusal registration.

The imprint or the registration of the maxillary shall be used for orienting the upper cast to the Articulator. No concern is to be given to the mandible as its function in this instance is ONLY to hold the Bitefork securely against the maxillary arch.

6. Figure 9 depicts the stem of the Bitefork entering the loose Bitefork Clamp. Slide this Clamp ① onto the stem perhaps 4 centimeters and then “Spring open” the Bow ② and swing it downward as ③ to allow the Earpieces to enter the the external auditory meatus.

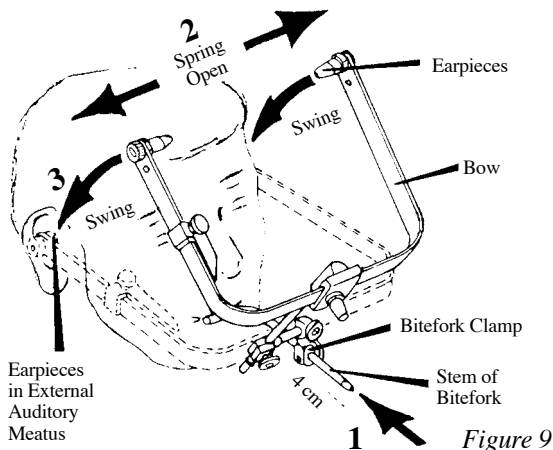
Figure 9

7. Slightly loosen the spring loaded Thumbscrew that secures the Anterior Patient Reference, Figure 10. Carefully rotate this Reference (Orbitale Pointer) to a proximity with the patient's infra-orbitale notch.

Elevate the Bow at the Transfer Rod to align the top of this Reference with the patient's orbitale.

This patient landmark is the lowest point on the infra-orbitale rim and is the anterior reference of the Frankfort Horizontal Plane.

The top of the Earpiece relates against the portion which is said to be the posterior reference of the Frankfort. The superior surface of the temple portion of the Bow now visually represents the true Frankfort Horizontal Plane.

8. Tighten the Clamp Thumbscrews in the suggested order, grasping the Bow to offset torquing and patient discomfort. A trial run of these Clamps is suggested before the patient application to recognize the amount of tightening necessary.

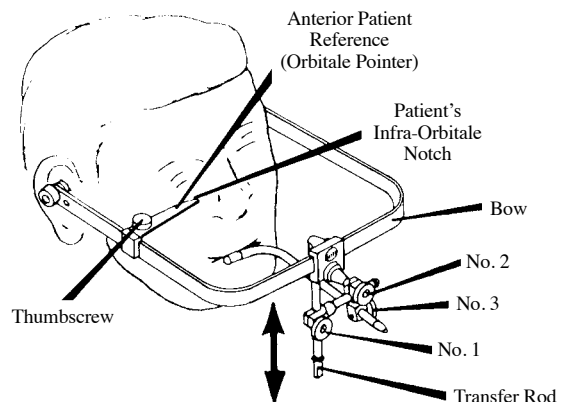
Figure 10

- A. Thumbscrew No. 1 - hold the right side of Bow while securing the Transverse Clamp to the Transfer Rod.
B. Carefully rotate the Anterior Patient Reference away from the patient. Store this Reference by swinging it forward to the right temple of the Bow and lightly engage the Thumbscrew.
C. Thumbscrew No. 2 - hold the right side of Bow while tightening the Bitefork Clamp to the Transverse Rod.

- D. Thumbscrew No. 3 - continue to hold the right corner of the Bow and tighten the Bitefork clamp to the stem of the Bitefork.
9. Instruct the patient to "open," releasing the Bitefork registration from the maxillary dental arch.

Grasp the posterior ends of the Bow and release the Earpieces from the meatus. Withdraw the complete assembly from the patient with the Bitefork index rigidly attached to the bow.

10. Recognizing that the Spring-Bow may be compromised by inadequately tightened Thumbscrews, Whip Mix offers an accessory Allen Wrench No. 005125-000 to preserve their integrity.

During patient recording OR immediately upon removal of the Bow assembly from the patient, the Thumbscrews may additionally secured by a Wrench. Tighten the socket screw head of the Thumbscrew in the absolute order of No.1, No. 2, and No. 3 illustrated, Figure 11.

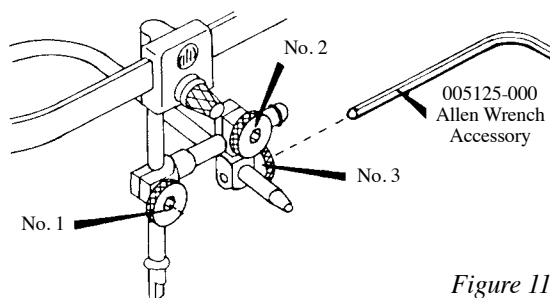
Figure 11

DIRECT-MOUNTING HANAU™ SPRING-BOW

LABORATORY PROCEDURE: The patient related HANAU™ Spring-Bow is consigned to the laboratory and is to be used for anatomically mounting the maxillary cast onto a HANAU™ Articulator.

SPRING-BOW SUSPENSION

11. The Earpieces represent the external auditory meatus and they have small vent holes in their ends. These Earpieces suspend the Bow directly over the Auditory Pins on the HANAU™ 96H2 and HANAU-MATE® Series of Articulators, Figure 12A.

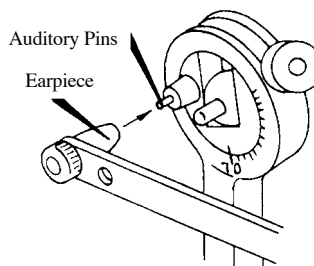
Figure 12A

A 5/32" diameter hole, 12mm anterior to the Earpiece center, represents the condyle center of the patient. The HANAU™ Wide-Vue, Radial-Shift and Modular Series of Articulators will specify removing the Earpieces and suspending the 5/32" dia. holes in the Bow over the Condylar Shaft Ends on the Articulator, Figure 12B.

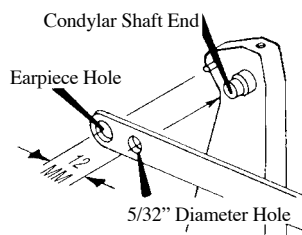
Figure 12B

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12. The Anterior Elevator, 010358-000, slides onto the Transfer Rod, Figure 13. Place the larger hole first and twist the Elevator over and onto the Rubber Ring, the Elevating Pin and Thumbscrew toward the front. Slide the Elevating Pin to its uppermost position and secure by Thumbscrew.

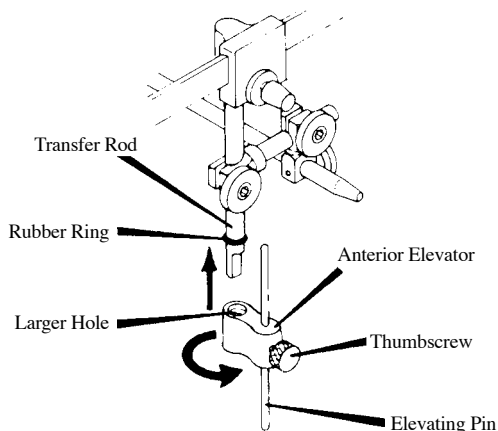
Figure 13

VERTICAL LANDMARKS

13. Some HANAU™ Articulators are furnished with an Orbitale Indicator as an anatomical simulation of the patient's infra-orbitale notch. The top of the Anterior Patient Reference (orbitale pointer) on the Spring-Bow shall be aligned to the underside of this Orbitale Indicator, Figure 14A.

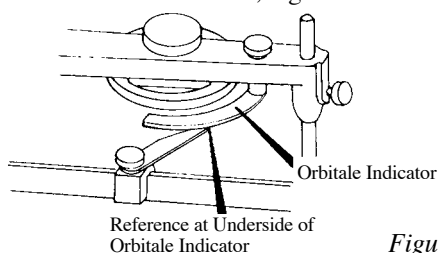
Figure 14A

Should the Articulator not be equipped with an Orbitale Indicator, the top of the Anterior Patient Reference will be aligned to an acceptable substitute—the bottom of the Mounting Plate, Figure 14B.

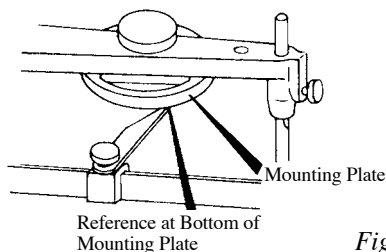
Figure 14B

The HANAU-MATE® has none of these references. A suitable substitute is to align the top of the Anterior Patient Reference about one and one-half millimeters (1/16") below the underside of the Upper Member, Figure 14C.

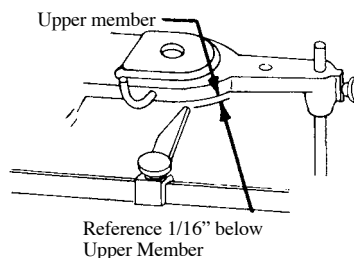
Figure 14C

MOUNTING THE MAXILLARY CAST

14. Rotate the Anterior Patient Reference to its storage position at the side of the Bow.
15. The Bitefork index must be adequately supported before placement of the maxillary cast and stone or plaster mounting to the Articulator.

An accessory HANAU™ Cast Support, Item No. 003401-000 is offered to support the Bitefork and maintain its position when adding the cast weight and stone mix, Figure 15.

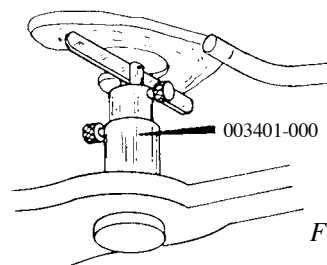
Figure 15

16. Seat and lute the maxillary cast in the occlusal index of the Bitefork, Figure 16.

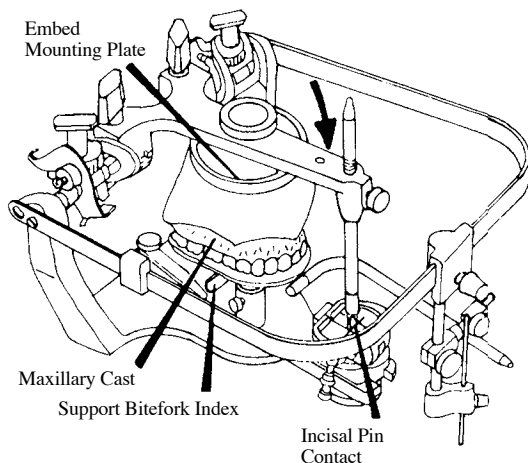
Figure 16

17. The upper member of the Articulator is swung open and the mixture of stone is placed on the maxillary cast. The Upper Member is immediately closed to embed the Mounting Plate and to bring the Incisal Pin into contact with the Incisal Table.

Complete the mounting, removing excess material to assure convenient removal and accurate reattachment of the cast to the Articulator.

18. Upon complete set of the mounting, disengage the cast from the Bitefork index and remove the Spring-Bow and supporting means.

The maxillary mount has been completed.

ADDITIONAL HANAU™ SPRING-BOW CAPABILITY

An accessory Mounting Platform/Cast Support, Item No. 010342-000 is available for INDIRECT MOUNTING of the maxillary cast, Figure 17.

The INDIRECT method allows the Spring-Bow to remain in the dental operator. The Transfer Rod/Biteplane assembly with casts or impressions and interocclusal records are sent to the laboratory for mounting. The maxillary cast is accurately related by the Transfer Rod placement in the Mounting Platform 010342-000. This Platform is used on HANAU™ Articulator Models as,

- ALL 96H2 Series
- ALL 158 Wide-Vue Series
- ALL 165 HANAU-MATE®
- ALL 166 Radial Shift
- ALL 181 Condyle Repositioner
- ALL 183/184 Wide-Vue Series
- ALL 190/192 Modular Systems


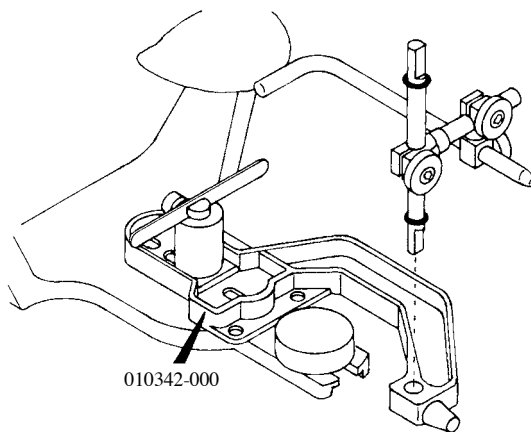
Figure 17

Consider an additional Transfer Clamp Assembly, Item No. 010356-000 and Bitefork 005726-000. A biteplane (perforated for index material adhesion), Item No. 005753-000 is recommended for occlusal cusp prints of the maxillary teeth. These additional adjuncts in effect become an additional Spring-Bow—doubling or tripling Bow use while the Bitefork index awaits later laboratory mounting.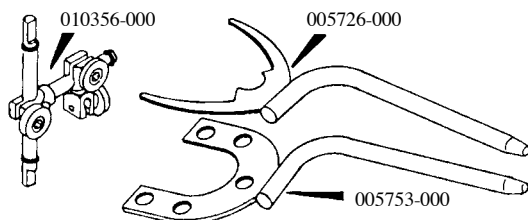

Additional adaptations to Articulators other than the premier HANAU™ line are described in the HANAU™ catalog and price list.

MAINTENANCE



Sterilization is accomplished by steam autoclave of the complete Spring-Bow and/or any part of it thereof.

Routinely, and especially after autoclave, apply a handpiece type of oil to all moving and threaded joints to assure free and continued movement.



STERILIZE BEFORE USE.

DO NOT DRY HEAT STERILIZE.



Articulator Preparation and Spring-Bow Attachment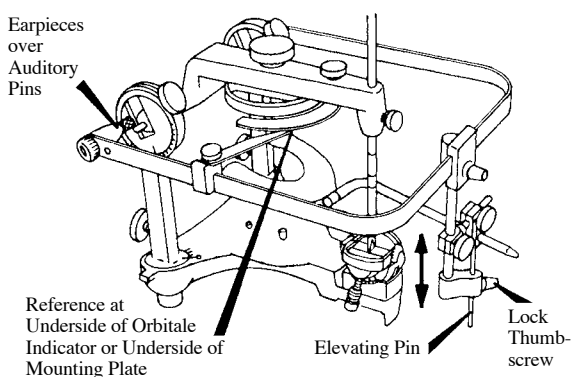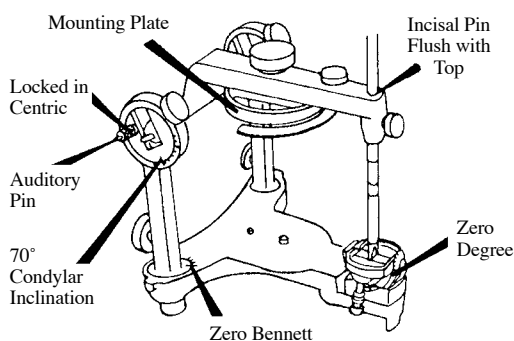



96H2 SERIES, serial 123,714 and above


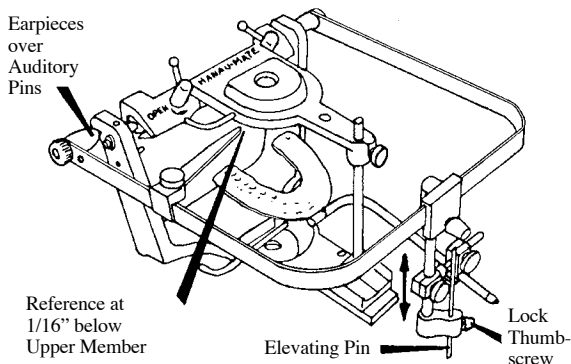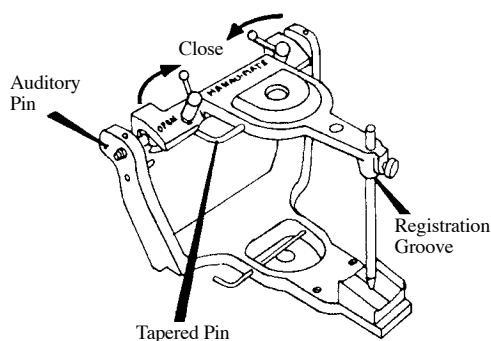
165 HANAU-MATE®


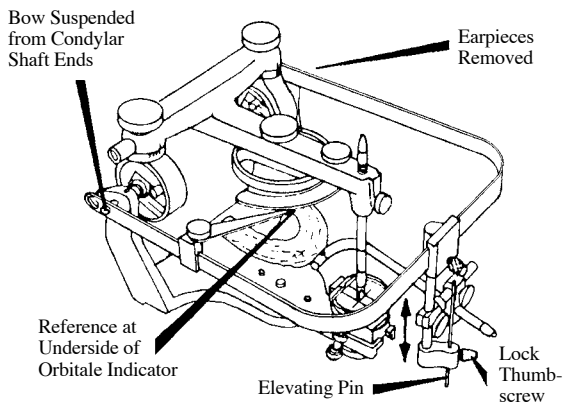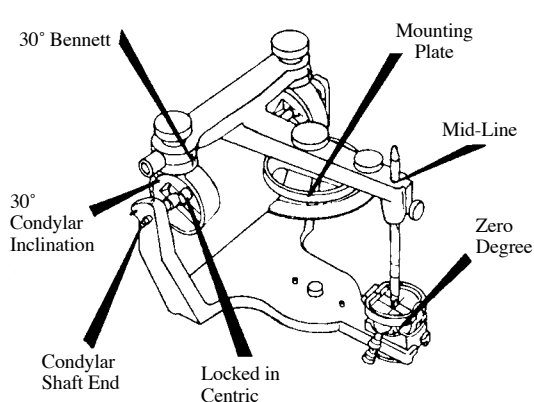
184/183 WIDE-VU SERIES

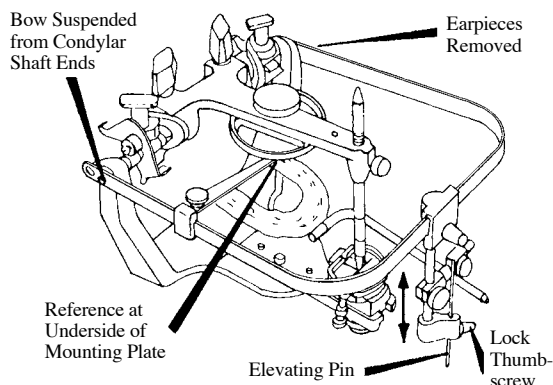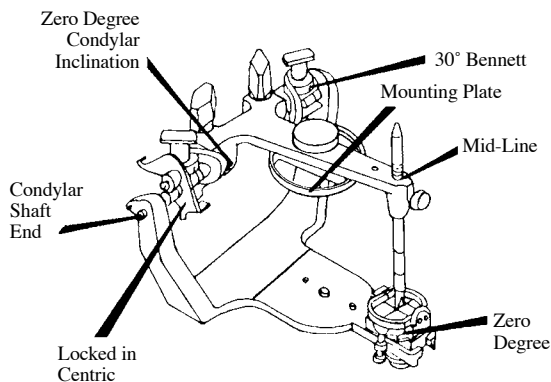190 MODULAR SYSTEM with Adjustable Bennett Fossae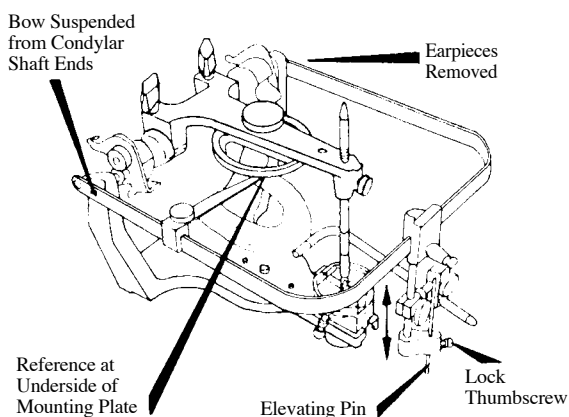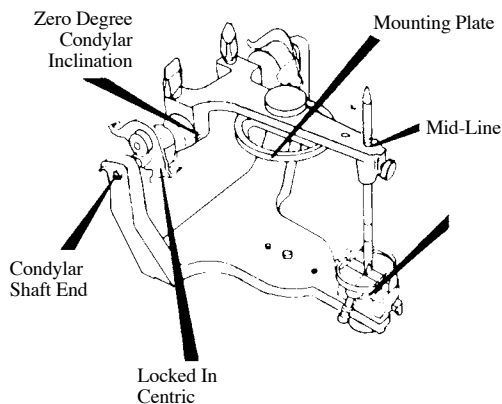

192 MODULAR SYSTEM with Programmed Fossae

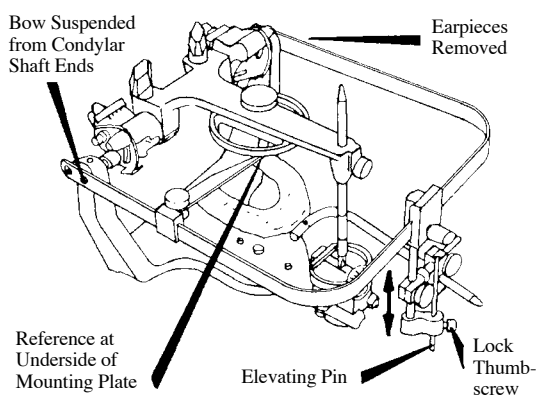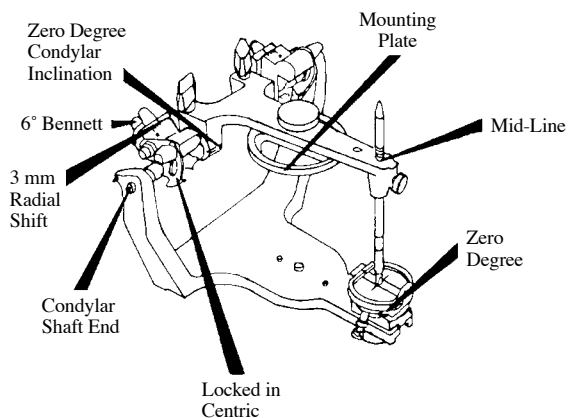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